

Carmellose Sodium

0.5% w/v



Every Carmellose Sodium 0.5% Unit contains sterile 0.5% w/v Carboxymethylcellulose Sodium free from any preservative, for single use.

INTENDED USE

The Carmellose Sodium (CMC) 0.5% is intended to use for lubrication of dry eye.

INSTRUCTIONS FOR USE

- Detach the container to be used.
- To open, twist and pull tab to remove.
- Instill 1 or 2 drops in the eye(s) and discard container.
- Not for multiple use. Discard the vial after single use.



1



2



3



4

STORAGE

- Store Below 25°C
- Do not Freeze
- Protect from (Sun) light
- Keep out of reach of children

PRECAUTIONS

- Do not use, if vial is damaged.
- Do not use, if solution is cloudy or discoloured.
- Do not touch the eye with the vial; this is to avoid eye infection or scratching.
- Not for treatment of any pathological eye conditions.
- Not to be used after the expiry date, marked on the package.
- Dispose off the vial as per the Country specific requirements.
- This product should not be re-used as it may cause microbial infection.
- Do not resterilize the final packed product as it is not suitable for resterilization.

Carboxymethylcellulose Sodium 0.5% w/v
Water for Injection q.s.



Manufactured by:

Sentiss Pharma Pvt. Ltd.

Vill. Khera Nihla, Teh. Nalagarh,
Distt. Solan, HP - 174101, INDIA

European Authorized Representative /
or EC REP:

Obelis s.a

Boulevard Général Wahis 53
1030 Brussels, BELGIUM

Tel: +(32) 2. 732.59.54

Fax: +(32) 2.732.60.03

E-Mail : mail@obelis.net



OBELIS S.A.
Bd. General Wahis 53
1030 Brussels, Belgium

CAE/17054/PI/GB/04
Revision Date : Aug 15